

Prontosan® Advanced Wound Cleaning

Instructions for use



1. Opening

Prontosan 350ml bottle

Remove the bottle top and dispose of the white safety ring, replace the bottle top and twist to pierce bottle then open.

Prontosan 40ml ampoule

Twist cap to remove.

Prontosan 30ml gel

Twist to pierce bottle then open.

2. Pre-soak

If existing dressing has adhered to the wound, then pre-soak with Prontosan Solution to aid / ease removal.

3. Irrigate

Irrigate wound to remove loose surface debris.

4. For even better results

- Soak a gauze swab in Prontosan, place on the wound and leave for 10 – 15 minutes.
- A thin layer of **Prontosan Gel** can be applied to the wound to provide continual, safe, gentle and effective debridement.

Hints & tips

- Date the Prontosan 350ml bottle and the 30ml gel when opened, as it can be used up to 8 weeks for single patient use.
- Prontosan Solution can be warmed to body temperature

Size	Unit / Pack	Product Code	NHS Logistic Code	Pip Code
350ml	10	400403	ELY248	324-8572
40ml	6	400412	ELY247	324-8580
30ml Gel	20	400505	ELZ213	324-8598

Available on FP10 and Scottish Contract.

FREEPHONE NUMBER 0800 093 0100

B | BRAUN
SHARING EXPERTISE